

Instructions for Use

ChicagoTip™ Sterile, Single Use, Frazier Self Cleaning Suction Tip

DEVICE DESCRIPTION

The ChicagoTip™ Frazier suction instrument is built with a self-contained obstructions removal mechanism (self-cleaning) during surgical procedure.

INTENDED USE/PURPOSE

This suction device is indicated to evacuate fluid or debris from the surgical site.

INTENDED USERS

The ChicagoTip™ is only intended to be used by persons trained in suctioning procedures.

INSTRUCTIONS FOR USE

1. Tear shaped cutout on the handle allows better control of the suctioning by varying the pressure.
2. If clogging occurs, push the stylet forward to remove the obstruction, then pull the stylet all the way back to its position. If the obstruction remains, retract the slider, disconnect suction tubing, insert the rescue stylet into the proximal end of the handle and advance it towards the tip.
3. Active suction may be resumed by retracting the stylet and reconnecting the suction tubing.

PRECAUTIONS

For Single Use Only. Do not reuse, reprocess or re-sterilize. Reusing, reprocessing or re-sterilizing may lead to patient and staff injury, infection or death due to device failure or infection.

Excessive retracting force can cause the device to break



Neuro Enterprises LLC
11 E Walton St.
Suite 3401
Chicago IL 60611 USA
+1 (312) 859-6889



Catalog number



Batch code



Use by



Do not use if package
is opened or
damaged



Manufacturer



Single Use



www.neuroenterprises.com



Consult
instructions for
use

Do not Resterilize



Sterilized using
Radiation



CE marking Conformity
European Notified Body
0297



Healthcare
professional use
only



Manufactured in
USA



Medical Device



Authorized
representative in the
European Community



Authorized
representative for
Switzerland